SIARETRON 1000 IPER

SIARETRON 1000 IPER | HYPERBARIC ELECTRONIC LUNG VENTILATOR

SIARETRON 1000 IPER the new electronic lung ventilator. Now achieving its 5th generation has been expressly conceived for any hyperbaric application.

Main feature:

- user-friendly
- respiratory rate stability at any depth
- volume automatic compensation
- Oxygen concentration adjustable from 21 up to 100%
- Dedicated program for newborns ventilation from 2 ml and up to 3000 ml for adults.
- Compatible with premixed HELIOX
- Extensive monitoring of any respiratory parameter including compensated volume and O₂ concentration
- Available with remote control panel for monoplace chamber installation and VGA connection for slave monitors outside the chamber
- Small portable and light (5.5kg)
- Both the menu design and the parameters self-adjustment of this ventilator make its use simple and intuitive
- Respiratory rate compensation at any depth.

The use of a low power special circuit controlling the functions of the Ventilator grants the accuracy of the set parameters. This stability cannot be compared with pneumatic ventilators.

- Automatic volume compensation

The rate stability is not enough to grant the perfect operation of the ventilator since the variation of the gas density due to the compression, tends to decrease the tidal volume at the depth increasing. The electropneumatic module automatically modifies the flow according to the diving depth, which is measured by a special absolute pressure transducer.

In this way, the ventilator delivers a constant volume at any depth

- Max flow up to 240 l/min.
- Oxygen concentration adjustable from 21 up to 100%
- Automatic safety system able to reduce the oxygen concentration at depth increasing.
- Electric feeding .

Through low voltage power and special hermetic battery. Built-in power supply unit

- Gas feeding.

Air and Oxygen at 3 bar above the operating chamber pressure.

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SIARETRON 1000 IPER

- Matchless performance up to a depth of 60 mH₂O
- Automatic compensation of all ventilation parameters
- High performance intensive care ventilator
- Adult, Paediatric and Neonatal ventilation thanks to an adjustable minimum Tidal volume from 2 ml to 3000 ml
- PEEP up to 50 cmH₂O and Flow by leakages compensation, with NIV availability.
- 9" TFT LED display, five graphs display simultaneously
- Three loops available P-V; P-F; F-V
- Compatible with premixed HELIOX
- External remote control panel to facilitate the therapy inside mono-place chambers
- Small dimensions and light weight (5,5 Kg)
- Special hermetic battery

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