


Sanix 2

Sanitization filter

Filtro per sanificare



Sanix 2

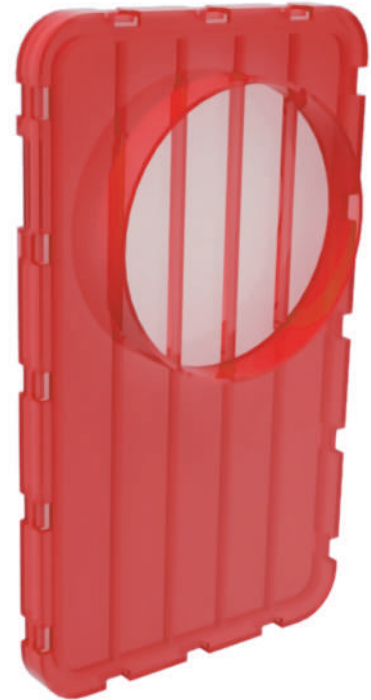

Code/Codice: SP - G00820000

Sanitization filter

Filtro per sanificare

Hospital infections and mechanical ventilation associated pneumonia (VAP) may represent a serious risk for fragile patients; in particular, ICU areas are frequently related to a variety of infection sources.

In order to prevent any potential hospital-related pneumonia, Siare designed **SANIX 2** sanitization filter, which can reduce the presence of *Pseudomonas Aeruginosa* by 94,08%. It is a simple and quick solution to sanitize intensive/sub-intensive and home care ventilators.

Materials and methods of use

Sanitization procedure consists in applying Sanix 2 where the air enters into the ventilator, which should then work for 20 mins. After the sanitization cycle is completed, the ventilator is ready to be used.

Sanix 2 is made by a plastic support which contains absorbing material, which, in turn, is soaked in 1 ml of sanitizing solution.

Sanix 2 sanitization filter is compatible with lung ventilators manufactured by Siare.

Le principali infezioni nosocomiali e in particolar modo la polmonite associata alla ventilazione meccanica (VAP) rappresentano un pericolo particolarmente significativo in terapia intensiva, dove i pazienti sono fragili ed esposti a molte possibili fonti di infezione.

Al fine di prevenire nel paziente il rischio di contrarre polmoniti nosocomiali Siare ha ideato il filtro sanificante **Sanix 2** in grado di ridurre del 94,08% la presenza di *Pseudomonas Aeruginosa*. La soluzione semplice e veloce per sanificare in modo efficace i ventilatori polmonari da terapia intensiva, subintensiva e domiciliare.

Materiali e metodi d'utilizzo

Il processo di sanificazione del ventilatore polmonare prevede l'applicazione di Sanix 2 in corrispondenza del sito di entrata dell'aria del respiratore e il funzionamento di quest'ultimo per 20 min. Al termine del ciclo di sanificazione il ventilatore polmonare è pronto per l'utilizzo.

Sanix 2 è costituito da un supporto in plastica all'interno del quale è contenuto del materiale assorbente imbibito di 1 ml di soluzione sanificante.

Il filtro sanificatore **Sanix 2** è compatibile con i ventilatori polmonari di produzione Siare.